



PRIMECARE[®]
Perimeter Cover
INSTALLATION GUIDELINES



INSTALLATION GUIDELINES

1. Thoroughly inspect prior to use for manufacturer defects. DO NOT USE if the Perimeter Cover is soiled or damaged.
2. Fit the mattress with the Perimeter Cover. Be sure to position the Velcro® end to the foot of the bed.
3. Secure cover to movable sections of the bed frame. Be sure bed can fully articulate and move up and down prior to use.
4. Make sure quick release buckles click into place and pull straps tight.
5. Metal clips are strategically placed at the bottom of the cover to secure a sheet if used.
6. A foot bolster may be attached to the Velcro® located at the foot end of the cover.
7. Prior to use, ensure the PrimeCare® Perimeter Cover is properly fitted to ensure the safety of your residents.

!WARNING!

This device is not a substitute for side rails or other protective devices for patients at risk of injury from falls due to unassisted bed exists. Always follow your facility's policies and procedures for patient assessment, monitoring and rehabilitation. Never use this product with patients in physical restraints with the side rails down. Serious injury or death could occur if patient becomes suspended off mattress surface.

DISPOSAL

If disposal is necessary, follow your facility's policies and guidelines for BIOHAZARD material.

CLEANING

Use mild disinfectants diluted to manufactures specifications consistent with mattress cleaning type products. DO NOT USE Benzyl or Phenol cleaners.

Foam sections are to be removed from the PrimeCare® Perimeter cover if using laundry services for cleaning.

STORAGE AND HANDLING

The PrimeCare® Perimeter cover is designed for indoor use only. If storage is necessary, make sure warehouse temperatures and humidity levels are normal. High humidity, extreme cold or heat may damage product materials.