



PRIMEPLUS™ TRAPEZE
INSTALLATION INSTRUCTIONS



INSTALLATION

TRAPEZE

Tools needed: 7/32" and 9/16" Hex Key)

NOTE: It is recommended that the Trapeze be assembled one side at a time.

- Mount the (2) Trapeze arms vertical by bolting them to the (2) mounting plates at the ends of the bed.
- These arms will be mounted on by (4) 7/32" Hex bolts
- After the bolts are tightened on both bars on both sides the cross bar can be assembled.
- The cross bar is going to be placed between the vertical arms.

NOTE: Make sure all bolts are facing the same way.

- Use the (2) 9/16" bolts to attach the cross bar to both arms. Finger tighten the locknuts until all (4) sides are up.
- Tighten the 9/16" bolts with wrench or socket.